Connexx

Version 9.8 and higher

User manual

English	5
Deutsch	14
Français	23
Italiano	32
Nederlands	41
Vlaams	50
Dansk	59
Norsk	68
Svenska	77
Suomi	86
Íslenska	95
Español	104
Português	113
Portugues Brazil	122

Ελληνικά	131
<hr/>	
Türkçe	141
<hr/>	
Eesti keeles	150
<hr/>	
Latviešu valodā	159
<hr/>	
Lietuvių kalba	168
<hr/>	
Русский	177
<hr/>	
Български	186
<hr/>	
Čeština	195
<hr/>	
Slovenský	204
<hr/>	
Slovensko	213
<hr/>	
BHS – Bosna i Hercegovina/Srbija	222
<hr/>	
Hrvatski	231
<hr/>	
Polski	240
<hr/>	
Magyar	249
<hr/>	

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Content

EN

Important information	6
Intended use	6
Personal safety	6
Recommended system requirements	7
Explanation of symbols	7
Conformance information	7
Getting started in Connexx	8
Screen layout	8
Online user guide	8
Client data and audiograms	9
Fitting workflow	10
Simulation	10
Hearing aid selection and detection	10
First Fit	11
Basic Tuning	12
Fine Tuning	12
Documentation	12
Additional settings	12

Important information

Intended use

The fitting Software Connexx is a tool intended to adjust programmable hearing aids according to the needs of people with hearing loss. The fitting must be performed by a hearing health specialist, e.g. audiologists, acousticians or ENT (ear, nose and throat) doctors.

Read and follow the instructions of this user guide to avoid injuries to the patient or damage to the hearing aids.

Personal safety

WARNING

Risk of impairing a client's residual hearing

Remote fitting should only be used for people who are fully able to answer questions concerning their hearing correctly. Therefore, remote fitting is not suitable for children under the age of 12 years or people with a learning disability. When fitting hearing aids via remote session:

- ▶ Verify if you are connected with the right client.
Example: Ask for the client's name and date of birth or start a video communication.
- ▶ Verify that the correct hearing aids are used.
- ▶ Instruct your client to wear the hearing aids correctly.

NOTICE

To avoid the loss of confidential patient data:

- ▶ Protect the system with security software, e.g. anti-virus software and firewalls.
- ▶ Keep the system and security software up to date. Install all software updates that are recommended for your system.
- ▶ Never leave an unsecured system unattended. Passcode protect the system and enable the screen lock feature to activate after a few minutes of inactivity.
- ▶ Do not connect unknown mass storage devices like USB sticks or external hard drives.
- ▶ Do not allow your system to connect to unknown wireless networks. These networks could be rogue access points that capture information passed between your system and a legitimate server.

Data privacy is an important element in looking after your patient's personal information. We care about our customers and would therefore like to offer our support in helping you protect your working environment from data loss or inappropriate access to patient data. Remember, as the data controller, it is your responsibility to comply with all local privacy laws, so this means ensuring only your authorized staff are able to use the Connexx software. If you need any help in protecting your environment, please do not hesitate to contact our data protection officer, who will be glad to assist you further.

Recommended system requirements

Operating System	Windows 10 with latest updates
Display	1920*1080
HW platform	Intel Core i5 (or similar compatible processor)
Free Hard Disk Space	free HD space > 10 GB
RAM	4 GB
USB interface	USB 2.0 or higher
Noah Version	Noah 4 (latest version)

Explanation of symbols

Symbols on the device or packaging

	CE compliance label, confirms compliance with certain European Directives, refer to section „Conformance information“.
	Medical device


Conformance information

The CE mark indicates conformity with the European Regulation (EU) 2017/745 concerning medical devices.

Getting started in Connexx

Screen layout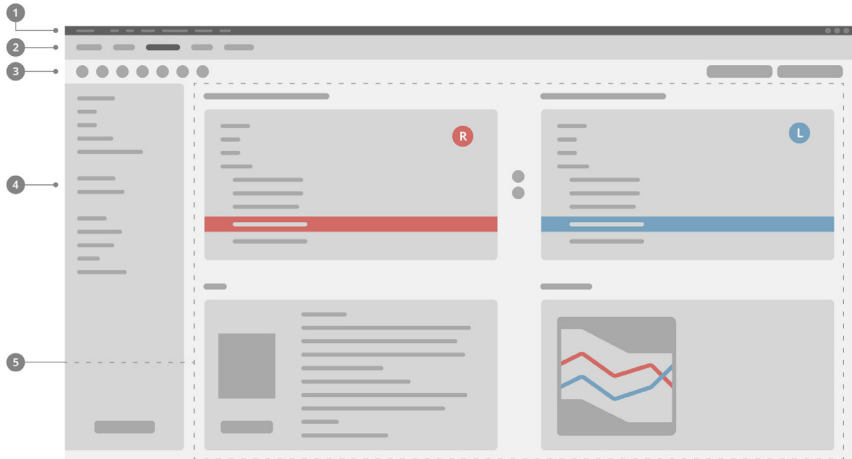

- ① Menu: Offers access to all functions.
- ② Workflow pages: Easy access to single workflow steps.
- ③ Toolbar: Is customizable to reflect the preferred functions (menu **Preferences > Edit User Preferences > General**).
- ④ Filter function: Enables the selection of hearing aids according to certain filter criteria.
- ⑤ Work area: Offers functions for the current workflow step.

Online user guide


This user guide is also available as a PDF document:
menu **Help > Electronic User Guide**.


Client data and audiograms

How you enter client data and audiograms depends on whether you start Connexx in standalone mode or via an independent database (e.g. NOAH).

Standalone mode

When you start Connexx in standalone mode, enter client data and audiogram manually:

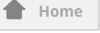- ▶ On the portal **Home** page , click **Load client**  and select a client or enter a new client profile.
- ▶ Load the client and proceed to the workflow page **Audiogram/Diagnostics**.
- ▶ To enter the audiometric data, double-click on each value across the frequencies.
- ▶ Proceed to the workflow page **Hearing Aids**.

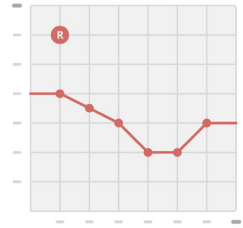
Via independent database

When you start Connexx via an independent database, the client's profile and audiometric data are automatically loaded. The audiogram is not editable in Connexx. Proceed to the workflow page **Hearing Aids**.

Fitting workflow


Simulation

Without having hearing aids connected:

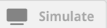- ▶ On the workflow page **Hearing Aids**, select hearing aids, check the fitting range and simulate a fitting by clicking the **Simulate** button .

Hearing aid selection and detection

- ▶ Connect hearing aids to the computer.
- ▶ If you use more than one programming device, select the programming device in the quick access area on the upper right.
- ▶ To detect the connected hearing aids, on the toolbar click the **Detect** icon .
- ▶ For **wireless** fitting: Assign the detected hearing aids to left or right side.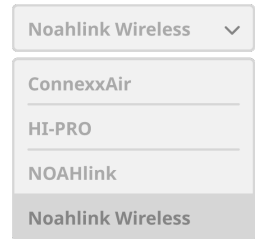

When programming wirelessly:


During the fitting process, the fitter is prompted to assign the serial number of a connected hearing aid to left or right side (serial numbers are printed on the housing or inside the battery compartment). The software displays the assignment of the side based on the selected serial number.

To ensure that the correct hearing aid is selected, a detection beep can be played.

- ▶ For **wired** fitting: The connected hearing aids are displayed.
- ▶ Select **New fitting** and proceed to the workflow page **Fitting**, starting with the workflow step **First Fit**.


First Fit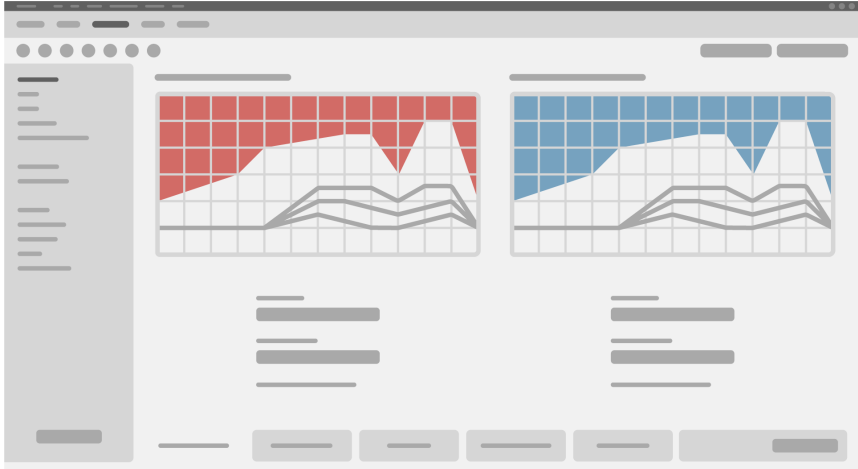

CAUTION

A high SPL output can be harmful for children's ears!

- ▶ Prior to direct evaluation of the hearing aid on the child, the hearing aid should be preset and tested in a hearing aid test box or real ear simulator. Use the child's RECD to verify that specified targets are met.

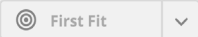- ▶ At the bottom of the workflow page **Fitting**, navigate through the tabs. Check and modify the proposed settings. For example, select the fitting strategy (manufacturer-specific fitting formula like AXFit, SmartFit, NAL-NL2, AS-Fit).
- ▶ To maximize feedback stability, an individual critical gain measurement (CGM) should always be run, because even when an individual CGM is not performed, an average critical gain curve is used in the background when calculating the targets.
- ▶ Once you are sure that everything is set to the patient's needs, click the **First Fit** button  to perform a First Fit.

- ▶ General preferences for the First Fit: menu **Preferences > Edit User Preferences > First Fit**.

Basic Tuning

On the workflow page **Fitting**, in workflow step **Basic Tuning**, adjust gain and loudness for, for example, loud sounds, medium sounds, soft sounds, speech, or adjust the sound quality (softer / sharper).

Fine Tuning

On the workflow page **Fitting**, in workflow step **Fine Tuning**, use the different tools to access different functions, for example sound settings, streaming settings, measurements settings.

More information on several functions in the software is available via tooltips. To display a tooltip, move the mouse cursor over the respective control.

Documentation

The workflow page **Documentation** provides an overview of hearing aid configuration, fitting parameters and listening program configurations.

- ▶ An individual fitting report can be configured for the selected client. The report can be opened and saved as a PDF, directly be sent via email or can be printed.

Under menu **Preferences >**

Edit User Preferences >

Print & Report, an individual logo or business card can be uploaded for use in the printout.

Additional settings

Volume control settings

On the workflow page **Fitting**, workflow step **Configuration > Hearing Aid**, you can read the current volume level, set the volume control level of the hearing aid when it is switched on, and set the range of the volume control.

Full-on-gain settings

For test-box measurements you can temporarily set the hearing aid's output level to full-on-gain.

Note that high levels can be harmful to your patient's ear.

- ▶ Ensure that the hearing aid is **not** in wearing position.
- ▶ Open a dialog box with test settings (menu **Fitting > Test Settings**).
- ▶ Switch on the **Full-on-gain setting**. The system warns you about high output levels.
- ▶ Read and follow the warning message.

When you confirm the warning message, the output level is set to full-on-gain.

- ▶ To quit this setting, switch off the **Full-on-gain setting**.

InSituGram specific restriction of output level

The output level is by default restricted to 100 dB HL.

You can deactivate this restriction under menu **Preferences > Edit User Preferences > Audiogram > InsituGram Specific Settings** in section **Restriction Of Output Level**.

Safety Limiter

The safety limiter restricts, by default, the output sound pressure level (OSPL) to 6 dB above the MPO. We do not recommend to change this setting. But if necessary, you can change the dB-value (menu **Fitting > Safety Limiter**).

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WSAUD A/S